

Trilogy200 ventilator

Clinician competency checklist

Clinician name: _____ Trainer name: _____ Date: _____

Goals	Date goals met	Trainer
<p>Clinician will be able to:</p> <p>1. Explain the buttons, indicators, and/or connectors on the front, rear, and side panels of Trilogy.</p> <p>Front panel</p> <ul style="list-style-type: none"> <input type="checkbox"/> Start/stop button <input type="checkbox"/> Key pad buttons <input type="checkbox"/> Backlight LED <input type="checkbox"/> Yellow alarm LED <input type="checkbox"/> Audio pause button <input type="checkbox"/> A/C power LED <input type="checkbox"/> Red alarm LED <p>Rear and side panels</p> <ul style="list-style-type: none"> <input type="checkbox"/> A/C power inlet <input type="checkbox"/> Exhalation porting block <input type="checkbox"/> Serial connector <input type="checkbox"/> Ethernet connector <input type="checkbox"/> O₂ connector <input type="checkbox"/> Detachable battery slot <input type="checkbox"/> Breathing circuit connection <input type="checkbox"/> SD data card slot <input type="checkbox"/> Remote alarm connector <input type="checkbox"/> External battery connector <input type="checkbox"/> Air inlet and filter <input type="checkbox"/> Cord retainer 		
<p>2. Describe all therapy modes and features.</p> <ul style="list-style-type: none"> <input type="checkbox"/> P/C <input type="checkbox"/> A/C <input type="checkbox"/> SIMV <input type="checkbox"/> S <input type="checkbox"/> T <input type="checkbox"/> P/C-SIMV <input type="checkbox"/> CV <input type="checkbox"/> CPAP <input type="checkbox"/> S/T <input type="checkbox"/> Ramp <input type="checkbox"/> AVAPS <input type="checkbox"/> Dual prescription <input type="checkbox"/> Flex comfort <input type="checkbox"/> Rise time <input type="checkbox"/> Sigh <input type="checkbox"/> Auto-Trak <input type="checkbox"/> Flow trigger 		
<p>3. Attach to A/C or D/C power and explain power source indicators.</p> <ul style="list-style-type: none"> <input type="checkbox"/> External battery <input type="checkbox"/> Detachable battery pack <input type="checkbox"/> Internal battery 		
<p>4. Attach to the O₂ source.</p>		

Goals	Date goals met	Trainer
Clinician will be able to:		
5. Assemble and connect the patient circuit. <ul style="list-style-type: none"> <input type="checkbox"/> Circuit configuration <input type="checkbox"/> Active with PAP (proximal airway pressure) <input type="checkbox"/> Passive exhalation port <input type="checkbox"/> Active with flow 		
6. Connect Trilogy to a remote alarm.		
7. View and change settings. <ul style="list-style-type: none"> <input type="checkbox"/> Access the startup and monitor screens <input type="checkbox"/> Navigate menu screens (full access and startup screens) <ul style="list-style-type: none"> <input type="checkbox"/> Safely remove SD data card <input type="checkbox"/> Settings and alarms <input type="checkbox"/> Options <input type="checkbox"/> Alarm log <input type="checkbox"/> Event log <input type="checkbox"/> Device information <input type="checkbox"/> Set circuit type <input type="checkbox"/> Enable the keypad lock feature 		
8. Access monitoring screens. <ul style="list-style-type: none"> <input type="checkbox"/> Detailed view off <ul style="list-style-type: none"> <input type="checkbox"/> Top monitor panel <input type="checkbox"/> Middle date/time panel <input type="checkbox"/> Bottom status panel <input type="checkbox"/> Detailed view on <ul style="list-style-type: none"> <input type="checkbox"/> Top monitor panel <input type="checkbox"/> Middle measured settings panel <input type="checkbox"/> Bottom status panel 		
9. Locate and explain alarm messages and alarm indicators <ul style="list-style-type: none"> <input type="checkbox"/> Explain high, medium and low priority alarms 		
10. Review care, cleaning, and maintenance of ventilator and access		

Clinician signature _____ Trainer signature _____

Respironics and Trilogy are trademarks of Respironics, Inc. and its affiliates.



© 2010 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: U.S. Federal law restricts these devices to sale by or on the order of a physician.

Philips Healthcare is part of Royal Philips Electronics

Europe, Africa, Middle East:
+33 1 47 52 30 00
Asia Pacific:
+852 3194 2280

www.philips.com/respironics

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668
+1 724 387 4000
+1 800 345 6443