
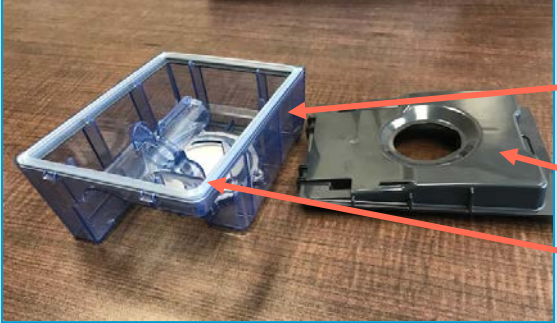

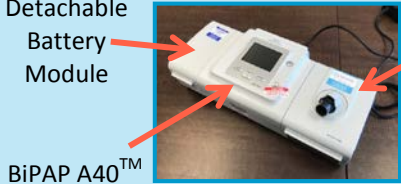


Item	Daily	Weekly	Monthly
Mask	Wash by hand with mild soap*. Rinse and air dry. Before use, be sure exhalation ports are not blocked.	*Mild soap is soap with NO emollients or antibacterial ingredients.	
Mask Cushion	Wipe with damp cloth and mild soap*. Rinse and air dry.	<i>Did you know?</i> You may only need to replace your mask cushion when you find that your mask no longer seals well.	
Head Gear		Wash by hand with mild soap* or place in a pillow case and wash in gentle cycle of washing machine with mild soap*	
Hose	 <p>Heated Wire Connection</p>	Remove tubing from humidifier and mask. Inspect for damage (cracking, tears, punctures etc) and replace if needed. Wash the air tubing <i>without getting heated wire connection wet</i> in warm water using mild soap*. Rinse and air dry.	<i>Heated Hose Part Number SYS1HT15</i>
Water Tank	Power OFF BiPAP A40™ . Slide the water tank out of the humidifier base (see page 2). Take the lid off the water tank and empty any remaining water. Wash the tank and lid with warm water and mild soap each day. Gently wipe the middle seal. Rinse and air dry. Instead of handwashing, you may wash humidifier parts in the dishwasher (top shelf).	In addition to daily cleaning, soak in a solution of vinegar and warm water (ratio 1 part vinegar to 10 parts water) for 30 minutes. Rinse and air dry.	 <p>Water Tank Base Water Tank Lid Middle Seal</p>
Air Filters	 <p>Reusable Gray Foam Filter Ultra-Fine White Filter</p>	Every 2 weeks, wash the gray filter by hand with warm water and mild soap*. Rinse well and allow to air dry. Replace every 6 months.	Do not wash the white filter. Replace monthly or sooner if dirty.
Machine	Be sure to unplug the power cord when you clean your equipment. Detach the battery module and remove the detachable battery before cleaning the module. Replace the battery once the module is completely dry.	 <p>Detachable Battery Module BiPAP A40™ System One Humidifier</p>	Wipe outside of the unit with a damp cloth and mild soap*



STEP 1

Turn System One™ humidifier and BiPAP A40™ OFF.
Allow heater plate and water to cool for about 15 minutes before cleaning.



STEP 2

Lift humidifier lid.



STEP 3

Grasp the top and bottom of the water tank to slide it out of humidifier base.



STEP 4

Push inner tab as shown to remove lid from water chamber.
Clean as directed on page 1.

Disclaimer

This information is provided to you for educational purposes only and should not at any time be considered for a substitute for professional advice from your physician or other qualified health care professional. The VEP does not assume any responsibility for application of the content of this document or for any information that may be present in the documents or websites cited as a reference. These references are provided for informational purposes only and do not represent VEP endorsement of any companies or products. Always consult your physician and/or a qualified health care professional to learn more about recommendations specific to your health needs.

References

1. Philips Respironics™ BiPAP A40™ User Manual
2. Philips Respironics™ System One™ Heated Humidifier User Manual