






Photos Courtesy Philips™ www.philips.ca/healthcare/product/HC1098159/cough-assist-e70-ventilation

Item	Daily	Weekly
Mask	Inspect after each use. If soiled, wash by hand with mild soap*. Rinse and air dry.	 <p><i>*Mild soap is soap with NO emollients or antibacterial ingredients.</i></p>
Tracheostomy Adapter	Inspect after each use. If soiled, wash by hand with mild soap*. Rinse and air dry.	 <p>A tracheostomy adapter is a standard 15 mm ID/22 mm OD connector.</p>
Tubing <i>Smooth Bore</i> <i>Non-corrugated</i>	After each use examine the tubing; if soiled, wash thoroughly with warm water and mild detergent, rinse well and air dry. Respironics Part Number RES532316	Remove tubing from bacterial filter at machine outlet. Inspect for damage (cracking, tears, punctures etc.) and replace if needed. Wash air tubing in warm water using mild soap*. Rinse and air dry.
Bacterial Outlet Filter	 <p>Always use a bacterial filter at machine outlet. Do NOT wash the bacterial filter.</p>	Replace outlet filter once a month or sooner if it is wet or soiled. Respironics Part Number RES20801
Air Inlet Filter	 <p>Air Inlet Filter Respironics Part Number 099477</p>	Wash in warm water with mild soap* and air dry. Wash every 2 weeks and replace every 6 months.
Machine	Turn the device OFF and UNPLUG the power cord before cleaning.	Clean the exterior of the device with a clean cloth dampened with water and mild soap*.

Helpful Hint

Some clients use the Cough Assist E70™ many times a day. It may be helpful to keep a few shorter sections of corrugated tubing available to trap secretions before they soil the long tubing. This shorter section of tubing may be thrown out and replaced between treatments.

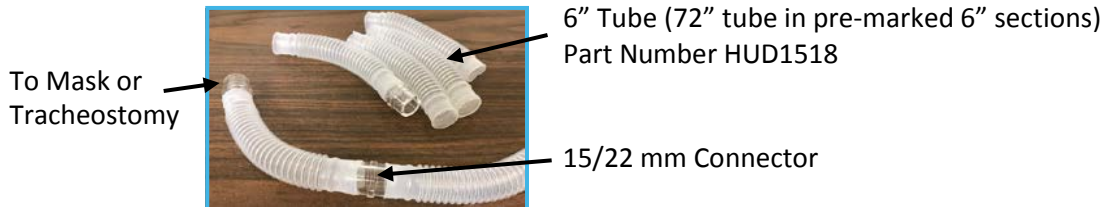

The smooth bore tubing, mask, 15/22 mm connector and bacterial outlet filter may be purchased as a kit.

Respironics Part Number RES1090833.

References

1. Philips Respironics™ Cough Assist E70™ User Manual
2. CANVent Canadian Alternatives in Noninvasive Ventilation <http://www.canventottawa.ca/>

Disclaimer

This information is provided to you for educational purposes only and should not at any time be considered for a substitute for professional advice from your physician or other qualified health care professional. The VEP does not assume any responsibility for application of the content of this document or for any information that may be present in the documents or websites cited as a reference. These references are provided for informational purposes only and do not represent VEP endorsement of any companies or products. Always consult your physician and/or a qualified health care professional to learn more about recommendations specific to your health needs.



640 Cataraqui Woods Drive | Unit 6 | Kingston, Ontario | K7P 2Y5
Tel: 613-548-6156 | Toll-Free in Ontario: 1-800-633-8977 |
Fax: 613-548-6157
Website: www.ontvep.ca